


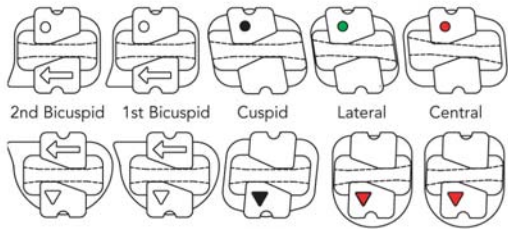



Circles identify the upper brackets and triangles identify the lower brackets.

Non-Extraction or First Molar Extraction



Four Bicuspid Extraction*

*Switch bicuspid brackets (side-to-side) when second bicuspids are extracted.

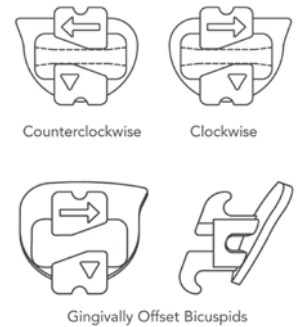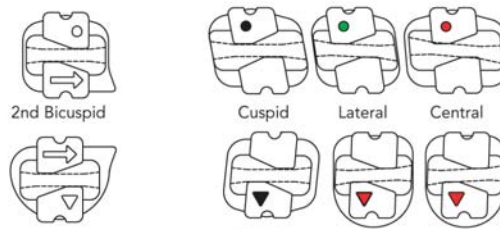

Arrows on the bicuspid represent the direction crowns should tip during initial treatment — either clockwise or counterclockwise. The bonding pad shape also points toward the direction of desired crown tipping.

Occlusal interference of mandibular bicuspid often leads to frequent rebonds. Gingivally offset mandibular bicuspid brackets are available with brackets positioned gingivally on bonding pads to permit proper bonding height on partially erupted teeth.

Positioning Jigs


Incisal Jigs are designed to help with accurate incisogingival bracket placement and assist with sighting long axis crown angulations. Jig measurements are from the center of the archwire slot to the incisal edge or cusp tips. Color coding identifies the various heights.



Universal Jigs are available without horizontal arms to enable custom bonding according to personal bracket height preferences. The increased occlusal length helps visually parallel the bracket with the long axis of the crown. Available in orange only.



Recommended Jig Heights

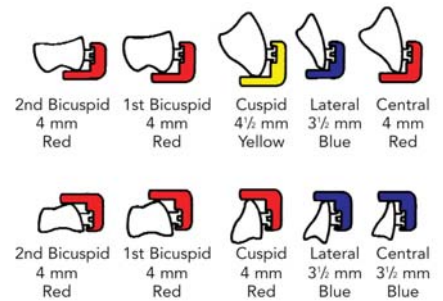

Jig height substitutions can be made for an additional charge. Green 5 mm jigs are also available for cuspids.

